(SV) Actimove® Clavicula Comfort

Kära kund,

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
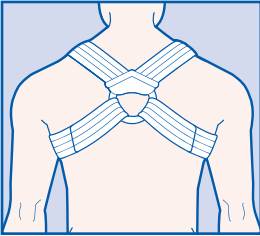
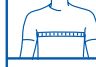
						
Maximum washing temperature 30°C (86°F) gentle process		Caution	Do not bleach	Do not tumble dry	Do not iron	Do not dry clean

	essity
---	---------------

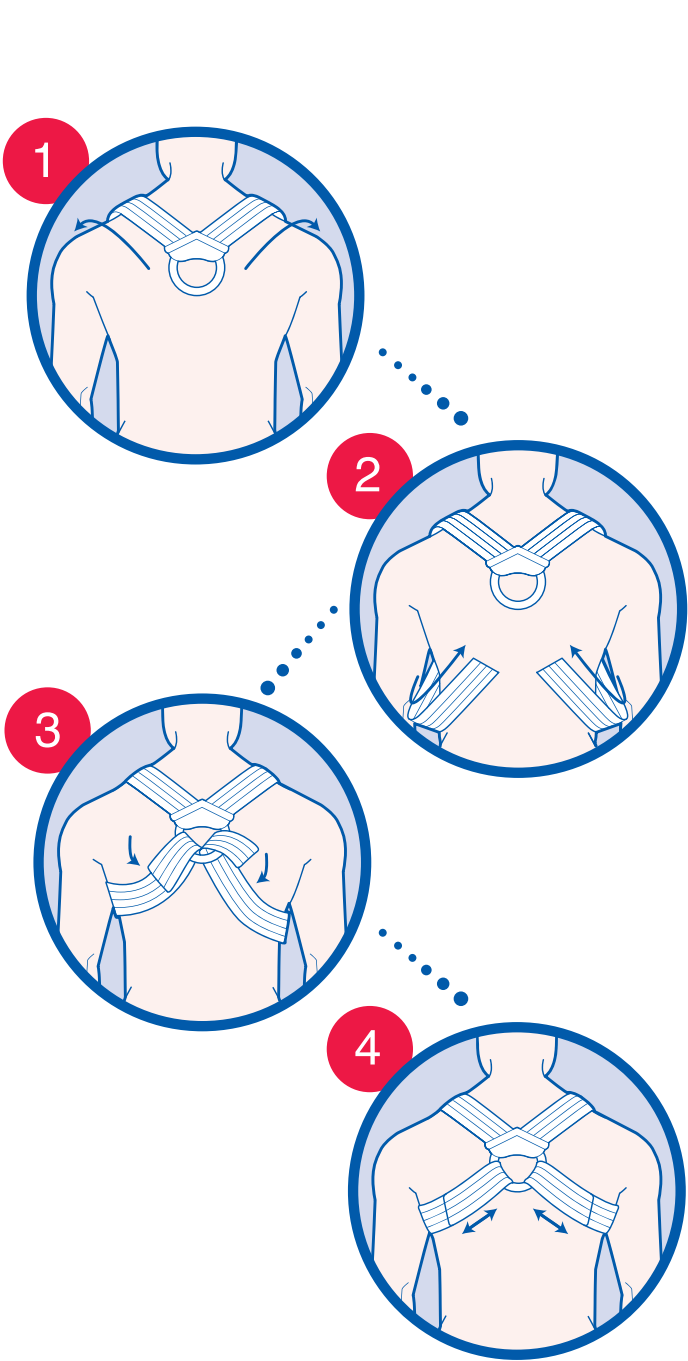Manufactured for:
BSN medical GmbH
Quickbornstrasse 24, 20253 Hamburg, Germany
www.bsnmedical.com

EN Actimove® Clavicula Comfort

Dear Customer,

Thank you for choosing Actimove® Clavicula Comfort.

Actimove® Clavicula Comfort is a functional and highly comfortable clavicle support. Its backpack construction pulls the shoulders backwards to align the clavicle fracture or to provide postural support. A flat plastic ring is positioned in the cross-section on the back and ensures a good anatomical fit. This construction enables the patient to feel comfortable even in a recumbent position. The rigid non-crease, non-stretch foam straps made of high tensile strap material provide the required stability while hook and loop fasteners allow easy readjustment and removal. The integrated soft cushioning axilla pad material is air-permeable and moisture wicking providing high wearing comfort especially in the axilla area.

Please follow the instructions and get well soon!

Intended use and indications:

Actimove® Clavicula Comfort is intended for support of the shoulder girdle.

Application fields include:

- Conservative treatment of clavicle (collarbone) fractures
- Postural support
- Postoperative use

Directions for Use:

Please note that Actimove® Clavicula Comfort must be applied for the first time by a healthcarepractitioner or trained specialist to ensure a correct fit and therapeutic support. You will also require assistance with removing and re-applying Actimove® Clavicula Comfort correctly.

- Place the clavicle support over the shoulders and around the neck. The ring should be located in the middle of the upper back and the Actimove® label should be facing outwards.
- Pull both ends from the front to the back under the armpits.
- Then pull the ends, beginning on the uninjured side, through the ring and secure tightly with the hook and loop closure.
- Now loosen both hook and loop fasteners and adjust the tension until equally tight on both sides. Secure the fasteners again when the optimum tension has been reached. Be careful not to pull the straps so tight as to interfere with the circulation or respiration.

Actimove® Clavicula Comfort is intended for single-patient use.

Contraindications:

Do not use Actimove® Clavicula Comfort in case of airway compression, persistent vascular and nerve damage or broken skin.

Cleaning & care:

Actimove® Clavicula Comfort is hand or machine washable (use the gentle wash cycle at 30 °C/86 °F). Close the hook and loop fasteners before washing. Please do not bleach, dry clean, tumble dry or iron and allow to dry completely before wearing again.

Notice:

Any serious incident that has occurred in relation to this device should be reported to BSN medical GmbH and the competent authority of your state.

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